

## Whitepaper

---

### Understanding Low Friction Coatings for Medical Devices

In the world of low friction, lubricious coatings for medical devices are diverse and complex. There are many different types of coatings, each with their own advantages, disadvantages, and applications. This article will provide a basic overview of the most popular medical device coatings and profile what can be expected from each type.

Lubricious coatings are most commonly distinguished by their degree of lubricity, or the amount of reduction in friction they provide. To understand what these measurements mean, we must first learn how these measurements are obtained. In the medical device industry, friction is measured by what is known as a Pinch Test. In a Pinch Test, a coated device is secured on an instrument that pulls the device between the jaws of a clamp which producing a load on the part. While clamped, the coated part is pulled at a fixed speed for a fixed distance, and the resistance to the pull is measured in grams of pull force. The grams of pull force needed to overcome the friction produced by the clamps divided by the clamp force yields the coefficient of friction (CoF).  $CoF = \text{Force}_{\text{pull}} / \text{Force}_{\text{clamp}}$



*Friction test system to measure lubricity*

A 500-gram clamp force ( $\text{Force}_{\text{clamp}}$ ) is a commonly used setting. The higher the pull force needed, the higher the CoF and the lower the lubricity of the surface. The lower the pull force, the lower the CoF and the higher the lubricity.

When measuring friction between two surfaces, another characteristic is that of static friction pull force vs. dynamic friction pull force. Static friction is the force needed to break the initial friction force so that the two surfaces can move relative to each other. Dynamic friction pull force is that force needed to maintain the motion once it has been initiated. Static friction (sometimes referred to as “stiction”) is almost always significantly higher than dynamic friction.

Now that we have an understanding of how the CoF is obtained and the significance of the different amounts of grams of pull force required, we can take a look at some of the commonly used coatings and compare them.

Before we look at the different types of coatings, let us begin by looking at the properties of an uncoated sample to provide a reference point. A very common material used in medical devices is PEBA<sup>®1</sup>, the brand name of polyether block amide. When uncoated, PEBA<sup>®</sup> (hardness of



55 ShoreD) generally produces pinch test results of about 500 to 600 grams of pull force. Assuming a 500-gram clamp force was used, the CoF would be (1.0-1.2), a very low level of lubricity. Other uncoated catheter materials of construction such as nylon 12, polyethylene and others exhibit similar CoF. Such a high level of friction would make many of today's catheter delivered minimally invasive procedures nearly infeasible.

*Catheter with lubricious coating*

Device manufacturers have turned to a variety of coatings to provide the surface lubricity not offered by the bare catheter material. The first class of coating is **Silicone oil (siloxane)**. This type of coating has long been used to reduce surface friction on medical catheters, introducers, etc. It is generally applied by the manufacturer, and is relatively inexpensive. It offers a much reduced CoF, at about 0.13, and approximately 125-175 grams of pull force. However, devices coated with silicone oil can be difficult to apply as they are very mobile and can spread to other manufacturing areas where they are undesirable. For example, the presence of silicone oil on a surface can prevent other surface enhancing coatings from adhering the device surface. Once silicone oil is present, it can be very difficult to clean from a device or work surface. As a result, these coatings can be a major contaminant risk to the manufacturer.

The second class of coating is **PTFE (Polytetrafluoroethylene)**, known best by the brand name Teflon<sup>®</sup>, is another popularly used material to reduce friction. It has been employed as a material of construction for catheter-type products or as an added coating for guidewires and

---

<sup>1</sup> PEBA<sup>®</sup> is a registered trademark of Arkema, Inc.




other devices made from metal. PTFE typically exhibits between 150 and 200 grams of pull force, or a CoF of about 0.3 - 0.4. On bare metals such as stainless steel and nitinol, PTFE offers performance similar to that of Silicone Oil. The main advantage of PTFE coatings is that they provide some level of lubricity while dry and do not require any source of moisture to perform.

**Hydrophilic coatings** are the third class of coatings employed to reduce surface friction and enhance lubricity. The term “hydrophilic” means that its water loving, in other words it readily wets out when exposed to moisture. Most hydrophilic coating consists of a loosely cross linked polymer coating that will readily uptake liquid when exposed to a source. This coating that surrounds the device acts like a microscopic sponge and the water (or other liquid) provides excellent low friction characteristics as long as the coating remains wet. Compared to PTFE coatings which are hydrophobic and repel water, hydrophilic coatings when wet provide superior lubricity (lower CoF). Devices coating with these coatings demonstrate pull forces as low as 3 grams and CoF values as low as 0.01. In clinical use, devices coated with hydrophilic material demonstrate superior control, exhibit lower stiction (static friction) behavior, and require far less force to perform any given movement. Ranging from 3-25 grams of pull force, hydrophilic coatings offer the best CoF, averaging between .01 and .05. This high level of lubricity enables devices to navigate tortuous anatomical pathways while reducing tissue irritation and adding to patient comfort.

Some types of hydrophilic coatings may generate particulates from their surface in use. For these applications where particle generation from a coating is critical, such as cardiovascular and neurovascular, the selected formulation must be one designed specifically for low particle generation. The only real limitation of this category of coatings is that they must be hydrated to perform.

Commercially available hydrophilic coatings such as Harland Medical Systems Lubricent® UV coatings average between 8 and 12 grams of pull force that reduces friction over an uncoated surface by as much as 98%. This type of low-friction performance increases the device’s ability to navigate through tortuous anatomical pathways, improves device control, reduces tissue damage and adds to patient comfort.

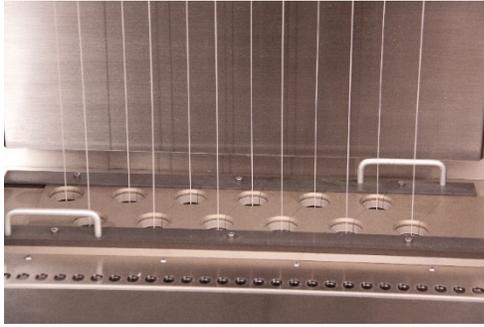

Advances in surface enhancements, especially low friction hydrophilic coatings, have accelerated minimally invasive surgery's growth. These coatings now impact a variety of surgical specialties, including cardiovascular, orthopedic, urological, peripheral and neurological procedures.

*Dip coating and curing process*

**Author**

Kevin Guenther

Methods Director

Harland Medical Systems

[info@harlandmedical.com](mailto:info@harlandmedical.com)